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Instructions Sheet

Remove all components from the box. Read all instructions carefully . For any queries contact on
email info@test2home.com or phone number **+971 6 5566 086**

Kit Content:

Item	Qt.	Item	Qt.
Barcoded test tube specific for you	1	Bio-bag contains Alcohol	1
UniStik Lancet	2	Pads, Plaster, Dry Cotton	1
		Additional Test menu	1
		Instructions Sheet	1

Sample Collection

Remove all items from the box. Take out the items found inside the biohazard bag

Clean your ring or middle finger by rubbing with the alcohol swab provided. Allow to dry.


Open the test tube cap and place the tube in a vertical position firmly in the designated hole.

Take out the single use Lancet. Turn the lancet cap several times until you feel it is separated. Gently pull the cover and dispose. **DO NOT FORCE PULL**

Place the Lancet firmly on your cleaned fingertip and press the button. The lancet will automatically prick the finger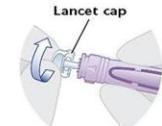

Point the finger downwards and allow blood to form a drop. If needed lightly massage the base of your finger to increase blood flow.

Collect approximately 1mL of blood (around 20 blood drops) into the test tube. Stop the bleeding by using the dry cotton .

Firmly re-cap the tube and make sure it is firmly capped (not leaking). Place the tube back again in its original place in the box. Call the lab for sample collection

Notes: Please make sure to collect enough blood sample to allow the lab to perform the tests requested. If more blood is needed, please prick a second finger using a new lancet "provided" using the same procedure. Make sure to tighten the cap of the tube upon closure to prevent blood leaks. Once collected, please call the lab for sample collection. Do not worry, your sample is stable for about 4 hours from the time of collection.